

# Instruction for use

## 1. Intended Use

The colonoscopy pants used for medical purposes in clinical facilities to extend the hygienic life of existing Scrubs of healthcare personnel in hospital and operating room environments and to prevent contamination from microorganisms such as viruses and bacteria and body fluids. This product is disposable and it prevents bacterial and viral transfer between the medical staff and the patient, reducing the risk of contamination and infection between the patient and the medical staff during the operation and patient visit.

## 2. Design

Endoaccess GmbH supplies colonoscopy pants in 2 different sizes (L and XL) applicable for most patients. Information on the range of sizes available may be obtained directly from us.

## 3. Before use

The user should study these instructions carefully before using the colonoscopy pants. In addition, the user must inspect the item for defects and damages after removing it from the pack. Do not use it in case of damage.

**The colonoscopy pants are supplied non-sterile by Endoaccess GmbH. The colonoscopy pants must be inspected before use. Only undamaged items must be used! Colonoscopy pants must be used only once!**

## 4. Types

Endoaccess GmbH offers one model of colonoscopy pants in one size.

Table 1:

No.	Model Name	Model spec.	Size	Colour
1	colonoscopy pants	EA-SCP-L	L	blue
2	colonoscopy pants	EA-SCP-XL	XL	green

Please refer to the main catalogue of **Endoaccess GmbH** for further information.

## 5. Product description

The colonoscopy pants are made of non-woven fabric with elastic waist. Please see picture as figure 1.



Figure 1 The colonoscopy pants

Table 1 The material list of colonoscopy pants

No.	Product components	Material	Material of body contact (Yes / No)
1	colonoscopy pants	Non-woven fabric	Yes

## 6. Cautions

- The colonoscopy pants are not sterilized by manufacturer
- The colonoscopy pants are for single use only and must be disposed of after use.

## 7. Operation

### 7.1 Check before use

- Open the packaging and check for damages. Do not use the item if damaged. Contact Endoaccess or your distributor for exchange.

### 7.2 Before the operation

The patient is dressed in the disposable colonoscopy pants prior to the operation to avoid inconveniences for the patient or delay of the procedure.

- Choose the right size of colonoscopy pants prior to operation

# Disposable Colonoscopy Pants

- Keep in mind that the hole with attached cover is only on the back side of the colonoscopy pants

## 8. Sterilization

- Colonoscopy pants are delivered non-sterilized
- Colonoscopy pants do not have to be sterilized before use
- For single use only.

## 9. Period of use

Three years

## 10. Storage

- Do not place any objects on top of the product or its packaging
- Do not store the products near aggressive chemical products
- Do not expose the items to direct or indirect sunlight or other ultra-violet rays

## 11. User

The user of this products must be specialized in the field of endoscopy. Users should have special training in handling and reprocessing of products for flexible endoscopy.

## 12. Complaints

In the interest of the health of our employees only unused items instruments will be taken back. These must be labelled as such, otherwise handling is not possible.

## 13. Legal Foundation

The Law of the European Union is applied.

## 14. Product Support

In case of questions or difficulties concerning our instruments please contact the local distributors or Endoaccess GmbH directly during regular working hours.

Address: Feldriethe 1, 30826 Garbsen, Germany

Phone +49 5131 4422 610, Fax +49 5131 4422 622

## 15. Symbols

	Non-Sterile		Keep away from sunlight
	Use by	<b>LOT</b>	Lot number
	Manufacturer		Date of manufacture
	Keep dry		Do not use if package is damaged
	Give attention to IFU	<b>REF</b>	Item number
	Do not resterilize		Do not reuse
		<b>MD</b>	Medical Device

## 16. Disposal

After use, dispose the instrument and packaging according to the regulations of the hospital or local waste law!



Endoaccess GmbH, Feldriethe 1, 30826 Garbsen, Germany  
info@endoaccess.de www.endoaccess.de

