

## Instructions for use

Dear customer:

Thank you for using our products. Please read the manual carefully before you use our products so that you can use and maintain the products accurately.

### 1. Intended Use

The disposable polyp trap is a commonly used instrument for collecting and separating polyps. For single use only.

### 2. Design

Endoaccess GmbH supplies two designs of disposable polyp traps suitable for most endoscopes.

### 3. Before use

Please study these instructions carefully before using the disposable polyp trap. In addition, also please inspect the instrument for defects and damages after removing it from the packaging. If damages or defects are detected in this inspection, the instrument must not be used on the endoscope and must immediately be returned to Endoaccess GmbH for inspection by an expert.

- Polyp traps are not sterilized on delivery
- Do not use if damaged.
- For single use only.
- To be disposed of after use

### 4. Types

Endoaccess GmbH offers disposable polyp traps in two different sizes.

Table 1: Model list

No.	Model name	Model Ref.
1	Disposable Polyp trap (4 chambers)	EA-SPT
2	Disposable Polyp trap (one chamber)	EA-SPTS

For further information regarding dimensions please refer to the **Endoaccess GmbH** main catalogue.

### 5. Product description

#### 5.1 Product structure

The polyp trap is composed of cover, body, and pipe. Please refer to illustration 1.



## Disposable Polyp Trap

### 5.2. Product Material:

Polyp traps are made of PE (Polyethylene), PS (Polystyrene) and Polypropylene)

No.	Product components	Material	Materials of body contact (Yes/No)
1	cover	Polyethylene	No
2	body	Polystyrene	No
3	pipe	Polypropylene	No

### 6. Cautions

- Only to be used by trained and specialized surgeons.
- Polyp traps are not sterilized by manufacturer
- For single use only; to be disposed of after use
- Keep away from moisture
- Only to be used with an endoscope
- Not to be used for any other purpose

### 7. Operation

#### 7.1

- Check the sealing of the pouch
- Check for damages or defects. Please contact Endoaccess GmbH or your distributor for exchange should any damages be detected during this inspection.

#### 7.2

- Please pay attention to compatibility between polyp trap and endoscope
- Firmly connect the first graded tube to the suction (vacuum tube)
- Now firmly connect the second tube to the endoscope (supply connection)
- Turn the lid of the trap to position "O"
- Now polyps can be sucked /collected. Place marking arrow on position 1,2,3,4 accordingly. Please collect only 1 polyp per chamber
- After each collection turn the marking back to position "O"
- After collecting all polyps remove the tube connected to the suction pump from the polyp trap. Then disconnect the other tube from polyp trap. Polyps can now be removed from the chambers and transferred to lab for further analysis

### 8. Sterilization / Cleaning

- Polyp traps are not sterilized by manufacturer
- Polyp traps do not need to be sterilized before use
- For single use only, to be disposed of after use

### 9. Period of use

Ten years

### 10. Storage

- Do not put any objects on top of the product or its packaging
- Do not store near aggressive chemical products
- Avoid direct or indirect sunlight or any other ultra-violet rays

Improper storage voids all warranty claims.

### 11. User

The users of instruments of Endoaccess GmbH must be specialists in their fields. products must be specialized in the field of endoscopy. He/She should have special training in handling and reprocessing of products for flexible endoscopy.

## Instructions for use

### 12. Repairing and complaints

- In the interest of our employees' health, only sterilized or disinfected instruments will be accepted as returns. They should be labelled as such; otherwise, they cannot be handled further
- Single use devices must not be repaired

### 13. Legal Foundation

The Law of the European Union is applied.



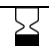




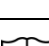

### 14. Product Support

In case of questions or difficulties concerning our instruments please contact the local distributors or the **Endoaccess GmbH** directly during regular working hours.

Address: Feldriethe 1, 30826 Garbsen, Germany

Phone +49 5131 4422 610, Fax +49 5131 4422 622

### 15. Symbols

	Non-Sterile		Keep away from sunlight
	Use by	<b>LOT</b>	Lot number
	Manufacturer		Date of manufacture
	Keep dry		Do not use if package is damaged
	Give attention to IFU	<b>REF</b>	Reference
<b>MD</b>	Medical Device		Do not reuse

### 16. Disposal

After use, dispose of the instrument and packaging according to the regulations of the hospital or local waste law!



Endoaccess GmbH, Feldriethe 1, 30826 Garbsen, Germany  
info@endoaccess.de      www.endoaccess.de

