

Instructions for use

1. Intended Use

The valve caps are used to protect and seal the opening of the biopsy channel of the endoscope.

For single use only

2. Design

Endoaccess GmbH supplies valve caps in standard sizes applicable for most endoscopes.

3. Before use

Please study these instructions carefully before using the disposable porthole cleaning brush. Please inspect the instrument for defects and damages after removing it from the packaging. If damages or defects are detected in this inspection, the instrument must not be used on the endoscope and must immediately be returned to Endoaccess GmbH for inspection by an expert.

- **Valve caps are not sterilized on delivery**
- **For single use only.**
- **To be disposed of after use**

4. Types

Endoaccess GmbH offers 2 different types of valve caps.

Table 1

No.	Model Name	Model spec.	Compatibility
1	Disposable Valve Cap	EA-SVC	Olympus
2	Disposable valve Cap Set	EA-SVC-S	Olympus

Please refer to the Endoaccess GmbH main catalogue for further reference.

5. Product description

Product Material: TPE (Thermoplastic Elastomer)

5.1 EA-SVC (compatible with Olympus endoscopes)



5.2 EA-SVC-S (compatible with Olympus endoscopes)



No.	Product components	Material	Materials of body contact (Yes/No)
1	Valve Cap (Disposable model)	TPE (Thermoplastic Elastomer)	No
2	Valve set (Disposable model)	TPE (Thermoplastic Elastomer)	No

6. Cautions

- a. To be used by professional operators only
- b. Not sterilized on delivery
- c. Only for single use, must be disposed of after use
- d. Keep away from moisture.

Disposable Valve Cap


7. Operation

7.1 Check before use

- a. The sealing of the packaging
- b. Open the packaging and check the instrument for damages. Do not use the item if damaged. Contact Endoaccess or your distributor for exchange.

7.2 Operation

- a. Insertion into the endoscope
- b. Please refer to the following illustrations displaying the operation of valve caps

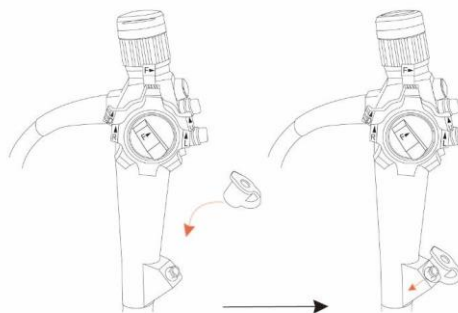

8. Sterilization

- a. Valve caps are not sterilized on delivery
- b. Valve caps do not need to be sterilized before use
- c. For single use only

9. Period of use

Ten years

10. Storage

- Do not put any objects on top of the product or its packaging
 - Do not store near aggressive chemical products
 - Avoid direct or indirect sunlight or any other ultra-violet rays
- Improper storage voids all warranty claims.

11. User

The user of this products must be specialized in the field of endoscopy. He/She should have special training in handling and reprocessing of products for flexible endoscopy

12. Repairing and Complaints

- In the interest of the health of our employees only disinfected or sterilized instruments will be taken back. These must be labelled as such, otherwise handling is not possible
- Single used instruments must not be repaired

13. Legal Foundation

The Law of the European Union is applied.

14. Product Support

In case of questions or difficulties concerning our instruments please contact the local distributors or the **Endoaccess GmbH** directly during regular working hours.

Address: Feldriethe 1, 30826 Garbsen, Germany

Phone +49 5131 4422 610, Fax +49 5131 4422 622

Instructions for use

15. Symbols

	Non-Sterile		Keep away from sunlight
	Use by	LOT	Lot number
	Manufacturer		Date of manufacture
	Keep dry		Do not use if package is damaged
	Give attention to IFU	REF	Item number
	Do not resterilize		Do not reuse
MD	Medical Device		

16. Disposal

After use, dispose the instrument and packaging according to the regulations of the hospital or local waste law!



Endoaccess GmbH, Feldriethe 1, 30826 Garbsen, Germany

